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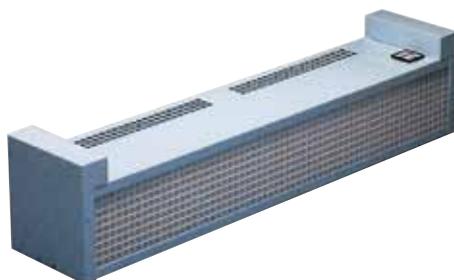
english Ultraviolet light possesses just the right amount of energy to break organic molecular bonds. As micro-organisms pass by the UV rays radiated from the ultraviolet lamp, this bond breakage translates into cellular or genetic damage for micro-organisms, such as germs, viruses, bacteria, fungi (like moulds), etc. This resulted in the destruction of the micro-organisms

REF : AF100.UV / AF200.UV / AF300.UV
ASEPTISEUR D'AIR FIXE / FIX AIR ASEPTISIZER

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english Each fix aseptisizers is equipped with an ultra violet source which emits in the germicidal band at 253,7 nm
Three products are available:

- **AF 100.UV** Germicidal protection barrier for installation above a door. Its two tangential turbines generate a vertical air flow of 300 m³/h. A time counter allows to monitor the operation time of germicidal UV lamps.
- **AF200.UV** Wall mounted appliance Its two tangential turbines generate a horizontal air flow of 300 m³/h. A time counter allows to monitor the operation time of germicidal UV lamps.
- **AF300.UV** Ceiling lamp to be fixed above a work bench. It must be used in the absence of personnel. Efficient for a volume of approximately 30 m³

MODELE / MODEL	AF100.UV	AF200.UV	AF300.UV
Longueur / Length	960 mm	960 mm	1024 mm
Largeur / Width	200 mm	210 mm	200 mm
Hauteur / Height	210 mm	200 mm	113 mm
Poids / Weight	15 Kg	15 Kg	4 Kg
Conso électrique / Power consumption	150 Watts	150 Watts	40 Watts
Durée germicide / Useful life	2000 h	2000 h	2000 h

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english This mobile device is developed for the air disinfection* of intensive care and operating theatre suites up to 100 m3.

It is equipped with eight ultra violet sources which emits in the germicidal band at 253,7 nm.

A fan settled in the center of the device allows constant air circulation in the room.

The treatment time is controlled by a timer. A time counter allows to monitor the operation time of germicidal UV lamps.

It can be used either in the absence of staff (opened shutters) or in a room in activity (closed shutters).

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* For the disinfection of surfaces the AM100.UV mobile air sanitizer can be coupled to the fog generator BIOFOG 200