LAVE-BASSINS BED-PAN WASHERS LPM 90

LPM 60

LPM 60

couvercle ouvert
bed-pan washer
with hatch open

LPM 90

couvercle fermé
bed-pan washer
with hatch closed

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*Thermal disinfection with
water softening system*

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LPM 60-CH

LPM 90-CH

Automatic bed-pan washers allow to empty, wash with high pressure, rinse and disinfect all types of basins, urinals and bottles.

Totally built in stainless steel 18/10 AISI 304. The sink is made of AISI 316.

EASY TO USE

- Front loading with:
 - fix support allowing all models of basins or round commode containers
 - 4 independent nozzles for urinals or bottles
 - 2 revolving nozzles
 - 1 cover handling device
 - accessories upon request
- Cycle starting operated through the front panel. The front control panel gives clear visual indications about:
 - cycle operation
 - temperature of disinfection (TH)
 - occasional dysfunction
 - number of cycles

EASY TO HANDLE

Microprocessor cycle management with user defined possibilities upon request:

- by prompt activation of the menu on front panel (Double washing, double rinsing, etc)
- software modification (cycle duration, automatic opening at the end of the cycle, etc)

HIGH TECHNICAL LIABILITY

- Technical components hardly tested
- Easy maintenance thanks to the front and lateral panels (Access to the main hardware components)
- Water softening device (TH)
- Electro valves with stainless steel protecting grill

SAFETY

- Cycle registration on paper thanks to a PC connection
- Lockable cover
- Emergency stop and reset

NOISELESS

Sound proof allowing a noise at maximum level 50 dB (A).

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EMPTIER:

Totally made of AISI 304 stainless steel with 400x400 mm pressed sink fitted with rinsing nozzles controlled by a quick-flow stopcock. Fold-down support grid for buckets and pots, completed with tap for filling them. Opening panel on the front with access to the drain coupling.

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WASHING UNIT:

The bedpan washers can be associated with emptiers, sinks and furniture in stainless steel. The hydraulic/electric connections between the various components are preassembled in the factory and do not require any particular work during installation.

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CONTROL DISPLAY UNIT 302

FOR FURTHER INFORMATION SEE THE BROCHURE ABOUT THE 302 M PROGRAMMABLE ELECTRONIC CONTROL UNIT

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OPERATING SAFETY

Involved control of the temperature and disinfection times.

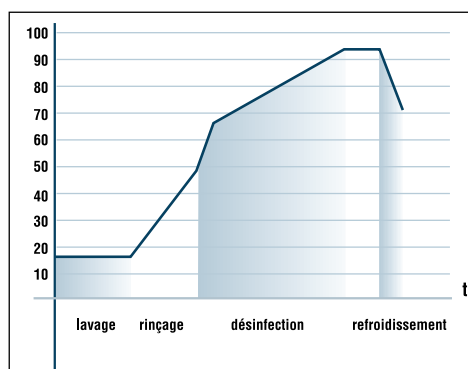
Hatch opening/closing system with electric lock that prevents it being opened during the cycle and microswitch that prevents enabling of the cycle when the hatch is open.

Audible alarm in the case of faults during the cycle (for SDTA).

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WASHING/DISINFECTION SYSTEM

Washing/disinfection system: the slipper bed-pans are washed by optimising the action of the rotating nozzles combined with that of the nozzles with conical jet and cutting washing action.

The disinfection cycle is done either through a chemical disinfecting liquid or by input of flowing steam. The temperature read in the chamber can be regulated between 80 and 95°C and maintained for an adjustable time.

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HYCARE MEDICAL consistently with its policy of continuous improvement on its own products, reserves the right to make changes at any time without prior notice. Therefore, all related data must be regarded as approximate and unbinding.

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